Diamond Universal

ENGLISH

Content:

1 syringe with 2 grams and 2 tips.

Product Description:

It is a universal polishing paste produced using micronized diamond with an extra fine granulation for dental or prosthodontist use only. The product offers excellent results in polishing and shining porcelain, dental enamel, resins, composites and other restoring materials. As it is soluble in water, it helps with the lubrication during the polishing and facilitates removal at the end of the treatment. It is a uniform, high hardness, good resistance product and its medium viscosity is not affected by heat.

Indication:

The Diamond Universal paste is suitable for finishing and polishing in surfaces of restoring materials in general.

Composition:

Micronized diamond powder, mineral oil, thickener and emulsifier.

Directions for Use:

1. Make sure if the packaging was not open, if so, neglect it.
2. With the help of sanding discs or medium-grain paste, prepare the region to be polished, letting it be smooth and in the desired format.
3. After the end of the previous procedures, place a small amount of the paste on a felt disc, or if preferred, directly on the tooth or restoring material and start polishing.
4. If you have not obtained the desired result, repeat the procedure.

Warnings and Precautions:

- Restricted use, for professionals who specialize in dentistry, in accordance with local legislation;
- Rigorously follow the procedures provided in the Use Instructions;
- To avoid overheating the surface, it is recommended to use low rotation and non-continuous movements in the application of the felt on the tooth;
- It is advisable to use new felts for the polishing process, in order not to compromise the final result of the procedure;
- Dispose of the tip after use;
- Do not let product come into contact with eyes, skin and clothing;
- Do not use the product after it has passed its expiration date.
- Keep out of reach of children.

Contraindications and Collateral Effects:

It should not be used in people with reported sensitivity to its components.

Storage and Transport:

Always store the product in its original packaging, at room temperature. Protect it from direct exposure to sunlight.

Method of Discard:

Discard the product in accordance with local legislation, assuring its complete disfigurement in order to prevent its reuse and damage to the environment.

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