14337 Total Blanc Home C - Manual Pac_Policro

84mm

DFL

TOTAL BLANC HOME C
CLAREADOR DENTAL

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TOTAL BLANC HOME C DENTAL WHITENING

PACIENT MANUAL
INSTRUCTIONS FOR THE PACIENT IN TREATMENT
English

Pacient: _____

Record the color before treatment: _____

Date: ____ / ____ / ____

Time of gel application: _____

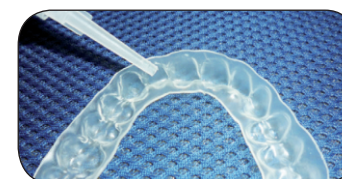
Carefully read the information below before using this product.

INSTRUCTIONS OF USE

1 - Floss and brush your teeth.

2 - Place the application tip to the syringe and put 1 drop of gel in space for each tooth in the tray. After use, cap the syringe.

3 - Place the tray on the teeth. Remove excess gel with cotton or with a toothbrush.

4 - Obey the residence time determined by the dentist.

5 - Remove the tray after the prescribed time. Wash and brush your teeth with toothpaste without dye and without sodium bicarbonate.

6 - Wash, dry and store the tray.

STORAGE AND TRANSPORT

Keep out of children's reach, at a place away from heat and direct light. Store the product between 5°C and 25°C. The product is valid up to 2 years after its manufacture date. Under conditions where the temperature is above 40°C, there may be product chemical degradation, with its efficiency loss.

CARE DURING TREATMENT

- Avoid smoking during the bleaching.
- Do not ingest acid food and sodas.
- Do not ingest products which contain colorant.
- Use white toothpaste without sodium bicarbonate.
- Do not chew or ingest food with the tray in the mouth.
- Avoid gel contact with the gum and skin. Apply in the trays only the necessary amount to cover the teeth, avoiding the excess of material. A drop per tooth is enough.

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DFL Indústria e Comércio S.A.