Praxis Mandrel

Use: Insert the mandrel shank into the chuck of the contra-angle handpiece. Install the finishing/polishing device on the mandrel. **Compatible with:** Praxis discs. **Connection with contra-angle:** The Praxis Mandrel is manufactured in compliance with specifications from the rule ISO 1797:2017.

Warnings, precautions and restrictions:

Professional use only. The use of the mandrel or discs with similar products is of the user's sole responsibility. To avoid an accidental swallowing of this product, it is necessary the application of a total isolation technique of the operator field. If there is proven allergy to any of its components, the product must not be used. Wear protective goggles to shield against particles (Dentists and patients). Use of a rubber dam is highly recommended to avoid the potential ingestion of dental materials or other debris. Rotating devices must be handled with care and control to avoid injury to the patient. Contact with soft tissues (skin, gingiva, mucosa) may result in injury to the tissue. Any serious incident related to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established. **Cleaning and Sterilization:** **Cleaning:** Carry out the cleaning with the aid of water and enzymatic detergent 4 enzymes (Amylase, Protease, Lipase and Carbohydrase) and with pure pH range between 6.5 - 7.5. Prepare the dilution solution according to the manufacturer's instructions. The water should be potable. For best effectiveness, it is recommended that the temperature be between 40°C and 55°C. Submerge the product and leave in contact for 5 minutes with the solution. Remove parts and rinse under running water for at least 10 seconds. Dry the piece with absorbent paper or disposable wipes. **Sterilization:** The mandrel should be autoclaved (126°C, 16min), after each patient. **Reprocessing Limits:** Repeated processing has minimal effect on the mandrel due to its composition. This is a reusable device. The manufacturer guarantees up to 240 reuses.

Inspection: Carefully inspect the mandrel to ensure that all debris has been removed. Observe damage / breakage of the product that would prevent proper operation and do not use in this case. **Composition:** Stainless steel. **Storage:** Store below 30°C (86°F), in a dry place, protected from light.

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Obelis S.A