## Superfix

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## Superfix

Superfix is a system for finishing and polishing composite resin restorations, consisting of abrasive discs that do not have a metallic center, allowing the use of the total abrasive surface. Available in 9,5 mm and 12,7 mm, and four different granulations codified by colors. It is used with Versaplex mandrel.

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**Use:** Install the finishing/polishing disc on the mandrel. Make sure that the disc is fully seated on the mandrel. Insert the mandrel shank into the hole of the contra-angle handpiece. Perform a test simulating the use to verify the product's effective rotation. For better results, use the four grits, successively from the coarsest to the finest one. Product must be applied dry, with low to medium speed and with intermittent touches. Discs of coarse and medium grits must be applied at 10,000 rpm rotation speed. Discs of fine and x-fine grits must be applied at 30,000 rpm rotation speed. Rinse and dry at each exchange of discs with different granulations.

**Connection with contra-angle:** Versapuls mandrel is manufactured in compliance with specifications from the rule ISO 1797:2017.

**Warnings, precautions and restrictions:** Professional use only. The discs are for single use only. Its reuse may cause cross contamination. The use of the mandrel or discs with similar products is of the user's sole responsibility. To avoid an accidental swallowing of this product, it is necessary the application of a total isolation technique of the operatory field. If there is proven allergy to any of its components, the product must not be used. Wear protective goggles to shield against particles (Dentists and patients). Wear a surgical mask to avoid inhalation of particles. Use of a rubber dam is highly recommended to avoid the potential ingestion of dental materials or other debris. Rotating devices must be handled with care and control to avoid injury to the patient. Contact with soft tissues (skin, gingiva, mucosa) is not in conformity to the tissue. The use of applied suction techniques applied during the procedure help limit the problem of particles discharging while using rotary instrumentation. Any serious incident related to the use of this product should be reported by the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

**MANDREL - Cleaning:** Carry out the cleaning with the aid of water and enzymatic detergent 4 enzymes (Amylase, Protease, Lipase and Carbohydrase) and with pH range between 6.5 - 7.5. Prepare the dilution solution according to the manufacturer's instructions. The water should be potable. For best effectiveness, it is recommended that the temperature be between 40°C and 55°C. Submerge the product and leave in contact for 5 minutes with the solution. Remove parts and rinse under running water for at least 10 seconds. Dry the piece with absorbent paper or disposable wipes.

**Sterilization:** The mandrel should be autoclaved (126°C, 10min), after each patient. **Repressing** **Use:** Repeated grinding of the same material effect on the mandrel due to its composition. This is a reusable device. The manufacturer guarantees up to 50 uses. **Inspection:** Carefully inspect the mandrel to ensure that all debris has been removed. Observe damage / breakage of the product that would prevent proper operation and do not use in this case.

**Composition:** Discs: Aluminum oxide, polyethylene terephthalate, synthetic rubber resin, water-based pigments. Fixing system: Polyvinyl chloride. Mandrel: Nickel plated metal.

**Storage:** Store below 30°C (86°F), in a dry place, protected from light.

Coarse Burgundy	Medium Pink	Fine Light pink	X-Fine White
-----------------	-------------	-----------------	--------------

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