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TOTAL BLANC OFFICE DENTAL WHITENING 35 % Hydrogen Peroxide English

In-Office Whitening PROFESSIONAL USE ONLY

TOTAL BLANC OFFICE is a 35% Hydrogen Peroxide whitening gel for dental whitening of vital and non-vital teeth, for dental office use only.

COMPOSITION

- Whitening gel: 35% hydrogen peroxide, thickener, plant extracts, amide, sequestering agent, glycol, dye and water.
- Neutralizer: sodium bicarbonate, preservative and water.
- Gingival Protector: diuretane dimethacrylate, camphorquinone, silica, tertiary amines, dye and essence.

MECHANISM OF ACTION

TOTAL BLANC OFFICE has as active principle the hydrogen peroxide, which mechanism of action is based on an oxidation chemical reaction. Hydrogen which breaks down to water and numerous byproducts including molecular oxygen, oxygen ions, hydrogen ions and free radicals to react with the pigments present in teeth, breaking the organic double bonds responsible for the dental structure darkening (chromophores).

PRESENTATIONS

TOTAL BLANC OFFICE (kit)
6 Syringes with 0.92g or 1 bottle with 5.6 ml of 35% Hydrogen Peroxide each
6 Syringes with 0.3g or 1 bottle with 1.8 ml of Thickener each
1 Syringe with 2.4ml of Neutralizer
1 Syringe with 3g of Gingival Protector
Accessories (application tips and connector)

TOTAL BLANC OFFICE (mini kit)
2 Syringes with 0.92g or 1 bottle with 1.9 ml of 35% Hydrogen Peroxide each
2 Syringes with 0.3g or 1 bottle with 0.6 ml of Thickener each
1 bottle with 2.4ml of Neutralizer
1 Syringe with 3g of Gingival Protector
Accessories (application tips and connector)

For replacement (refill):

- 1 Syringe with 0.92g or 1 bottle with 1.0 ml of 35% Hydrogen Peroxide
- 1 Syringe with 0.3g or 1 bottle with 0.3 ml of Thickener
- Accessories (application tips and connector)

STORAGE AND TRANSPORT

Keep out of children's reach, at a place away from heat and direct light. Store the product between 5°C and 25°C. The product is valid up to 2 years after its manufacture date. Under conditions where the temperature is above 40°C, there may be product chemical degradation, with its efficiency loss.

CONTRAINDICATIONS

- Pregnancy and breastfeeding
- Patients with periodontal diseases
- Patients with incomplete root formation
- Patients with allergic reactions to the product components.

GENERAL SAFETY INSTRUCTIONS

- The handling of the product shall be conducted exclusively by a professional.
- Wear gloves during the procedure.
- Shield patient's and professional's eyes with protective eyewear.
- Protect patient's lips with petroleum jelly or equivalent product.
- The content of syringes with peroxide may be under pressure and/or may have leaked due to the adverse storage conditions. Before open the package, protect your hands and eyes and observe if there is no liquid leaking inside.

- Avoid product contact with skin, soft tissues or clothing.
- Do not ingest the product.
- In case of accidental contact with skin or intra-oral mucosa, clean the area with running water. In such cases, the action of the gel can be neutralized by **TOTAL BLANC OFFICE NEUTRALIZANTE**.
- In case of contact with eyes, rinse immediately with running water and seek medical attention.

Product wastes must not be reused. They must be disposed as the local waste regulations.

PRECAUTIONS

Before Whitening

- Promote detailed anamnesis.
- The patient should be with the tooth surface thoroughly clean and without calculus.
- The gums should be healthy without inflammation.
- Perform prophylaxis with pumice and water, not using sodium bicarbonate.
- Perform calculus removal at least 2 days before the whitening beginning.
- If the patient has spontaneous sensitivity in the teeth, treat them before whitening.
- If the teeth to be whitened present gingival recession, isolate the area with **TOTAL BLANC OFFICE PROTETOR GENGIVAL**.
- Make sure that the teeth to be whitened do not contain caries or infiltrations on restorations.
- In case of teeth with a small fissure or incisal wear, protect the area with the **TOTAL BLANC OFFICE PROTETOR GENGIVAL** before applying the gel to prevent sensitivity.
- Gums surrounding teeth under treatment should be protected with **TOTAL BLANC OFFICE PROTETOR GENGIVAL**.

During Whitening

- Avoid gel contact with the soft tissues.
- In case of contact, apply **TOTAL BLANC OFFICE NEUTRALIZANTE**.
- If patient presents sensitivity, whitening can be discontinued after professional careful evaluation.
- After removing the gel, apply **TOTAL BLANC OFFICE DESSENSIBILIZANTE**, leave it for 3-5 minutes and polish over the **DESENSIBILIZANTE**.

CAUTIONS AFTER THE WHITENING

- Avoid smoking for at least 2 days.
- Do not ingest acid food and sodas.
- Do not ingest products which contain colorant for at least 2 days
- Use white toothpaste without sodium bicarbonate.

OPTIONS OF USE

TOTAL BLANC OFFICE may be utilized with or without light emitting equipment. If the option is to use it without light, follow the step by step. However, if the professional opts for the use of light with the bleaching, there may be a significant decrease on the procedure time. Additional information upon the technique with light emitting equipment may be obtained on www.dfl.com.br.

STEP BY STEP – VITAL TEETH


1 - Record the initial dental shade.


2 - Position the lip retractor and perform prophylaxis with pumice and water.

3 - Dry the teeth and the gingival tissue region for application of **Total Blanc Office Protetor Gengival**. Next, remove the syringe cap and fit the syringe applicator tip tightly.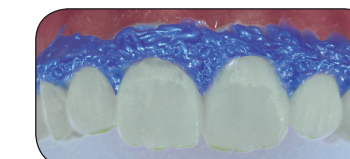

4 - Apply the **Total Blanc Office Protetor Gengival** in small layers covering around 0,5 cm of adjacent gingival tissue, sealing all the region to be worked, including gingival recessions.

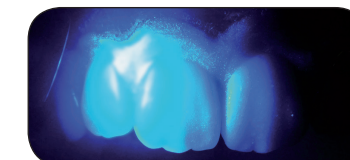
5 - Light cure **Total Blanc Office Protetor Gengival** for 30 seconds. Verify the hardening and adherence of the gingival barrier with a dental explorer.


6 - Kit with syringes: Remover the caps of the syringes with Peroxide and Thickener and attach the connector at the both syringes. Assure that the both syringes are well connected in order to avoid material leakage. Push the plungers alternately, 6 to 7 times, until the product reaches a homogeneous yellow color, indicating that it is active. Then, move all mixture into one of the syringes.

Kit with bottles: In the preparing plate, mix the Peroxide with the Thickener with assistance of a spatula until the product reaches a homogeneous yellow coloring, indicating that it is active. Always use the proportion of 3 drops of Peroxide for 1 drop of Thickener. For one arcade bleaching (10 teeth), it is recommended a mixture of 21 drops of Peroxide for 7 drops of Thickener.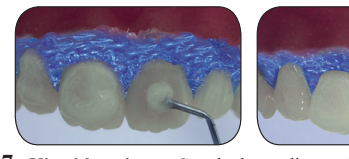


7 - Kit with syringes: Couple the applicator tip in the syringe and apply the bleaching gel over the teeth labial surface. The mixture content is sufficient for simultaneous application on the two arcades, on average.

Kit with bottles: Apply the bleaching gel over the labial surface of teeth with a spatula or brush. A gel layer with 1 mm of thickness must be applied over all the labial surface of teeth.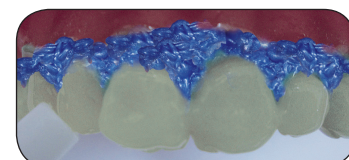

8 - The bleaching gel shall maintain contact with the tooth surface for 20 minutes. After this time, remove the gel with suction tip and clean the surface with moistened gauze.

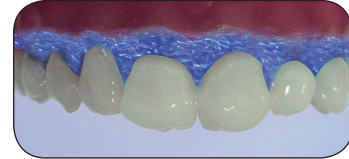
9 - Repeat steps 06, 07 and 08, completing 40 minutes of gel contact with the teeth. After that aspirate the gel with a suction tip and wash the teeth surface with abundant water.

10 - Remove **Total Blanc Office Protetor Gengival**.


11 - Apply **Total Blanc Office Desensibilizante** for 3 to 5 minutes.

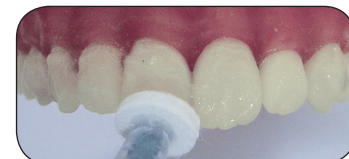12 - Polish the teeth labial surfaces using felt discs over the **Total Blanc Office Desensibilizante**.

13 - Remove **Total Blanc Office Desensibilizante** and perform final shade recording.

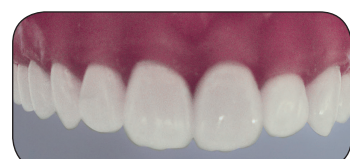
14 - If the desired color is not obtained after the first session, the procedure may be repeated after 7 days.

STEP BY STEP - NON-VITAL TEETH

1 - Record the initial shade of the devitalized tooth and adjacent teeth.

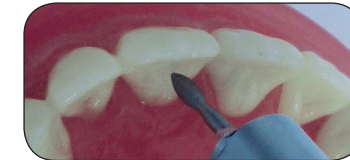2 - Access pulp chamber and remove gutta-percha up to 3 mm below the cementoamel junction.

3 - Make a peripheral sealing with glass ionomer, flow resin or zinc phosphate cement with 2 mm minimum thickness. Make sure the pulp chamber is free of residues.


4 - Apply the **Total Blanc Office Protetor Gengival** through labial and lingual/palatal faces with a thickness of 0,5 cm protecting all the area of the gingival tissue. Photopolimerize the gingival barrier for 30 seconds.

5 - Kit with syringes: Remover the caps of the syringes with Peroxide and Thickener and attach the connector at the both syringes. Assure that the both syringes are well connected in order to avoid material leakage. Push the plungers alternately, 6 to 7 times, until the product reaches a homogeneous yellow color, indicating that it is active. Then, move all mixture into one of the syringes.

Kit with bottles: In the preparing plate, mix the Peroxide with the Thickener with a spatula until the product reaches the homogeneous yellow coloring, indicating that it is active. Use a proportion of 6 drops of Peroxide for 2 drops of Thickener.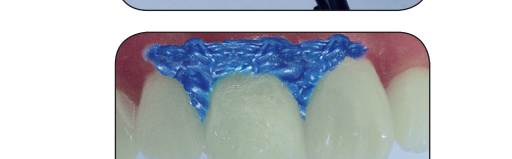

6 - Kit with syringes: Couple the applicator tip on the syringe and apply the bleaching gel with 2 mm of thickness on the labial face and inside the pulp chamber.

Kit with bottles: Apply the bleaching gel with 2 mm of thickness on the labial face and inside the pulp chamber with assistance of a spatula or brush. The gel must be applied 2 times in the same session and remain for 20 minutes in position for each application.

7 - After the second application, aspirate the gel and wash with abundant water.

8 - Remove **Total Blanc Office Protetor Gengival** and prepare the temporary restoration.

9 - If the desired color is not obtained after the first session, the procedure may be repeated after 7 days.

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DFL Indústria e Comércio S.A.