Poligloss

Pt

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Poligloss

En

Aluminum oxide-based polishing paste for general use in dental offices and dental laboratories. Poligloss provides an excellent gloss in polishing composite resins, amalgam, and dental enamel.

Use: Poligloss paste should be used with felt discs to enhance its action on providing the final polishing. Prepare the area to be polished, by modeling it and finishing it to achieve the desired anatomy and a smooth surface. Fit the disc into the mandrel. Apply the paste on the disc, or directly on the tooth or the restorative material. Apply the product using a low speed and applying low pressure, with intermittent strokes and circular movements. After the procedure, rinsing with water jet and suction is recommended.

Warnings, precautions and restrictions: Professional use. Single use only. Its reuse can cause cross contamination. The product should not be applied in patients allergic to any of its components. Wear protective goggles to shield against particles (dentists and patients). Wear a surgical mask to avoid inhalation of particles. Follow the instructions for use to prevent excessive heating of the surface. Keep the product tightly sealed in its original packaging.

Physical principle of operation: The discs absorb the polishing paste, and the friction with tooth surfaces and restorative materials provides the polishing action.

Storage: To be stored between 15°C and 30°C (59 °F and 86 °F), in dry place and protected from sunlight.

Composition: Polyethylene glycol, aluminum oxide, glycerin and flavoring agent.

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